**3M** Science.  
Applied to Life.™

# RiWaCS ATP proposal

(Rinse Water Collection System)

Dirk ERALY

30 mei 2017

## Letters to the Editor

### Variability of ATP amount in last rinse water of automated washer-disinfectors demands monitoring of every load

*Sir,*

Decontamination processes for medical devices, such as washing, disinfection, and sterilization are essential steps in

infection control. The end result of a washing–disinfection process is critical for the next step in decontamination, or for the safe use of a device on a patient in cases where no further decontamination is applied. It would be helpful for daily practice and the development of evidence-based standards if a data set were available to demonstrate the reproducibility of automated washer-disinfectors (AWDs).<sup>1,2</sup> We were unable to identify such a data set in the literature, and therefore decided to perform an inventory study. This encompassed 18 AWDs distributed over eight hospitals in four European countries. The cleanliness of a reprocessed instrument cannot be judged


# Rinse Water Collection System (RiWaCS)

## Components

3M™ Clean-Trace™ Online Software

3M™ Clean-Trace™ RiWaCS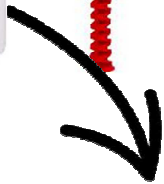


3M™ Clean-Trace™ NGi + ATP Water Test

# Rinse Water Collection System (RiWaCS)

How does it work

The funnel is so wide that water can only access the beaker via the funnel. The **RiWaCS** is used in an automated washer-disinfector (AWD). The stainless steel **RiWaCS** is constructed such that it does not tip over during an AWD process, e.g. by the force of a water spray.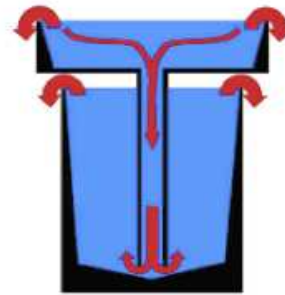

The red arrows indicate the water flow direction. During a process, water will be collected by the funnel and flow into the beaker. The funnel outlet extends almost to the bottom of the beaker. The bottom of the rinse water collector is constructed such that the water present in the beaker is pushed upwards and out of the beaker. Hence, at the end of a process the **RiWaCS** essentially contains the last rinsing water.

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
# Resultaten :

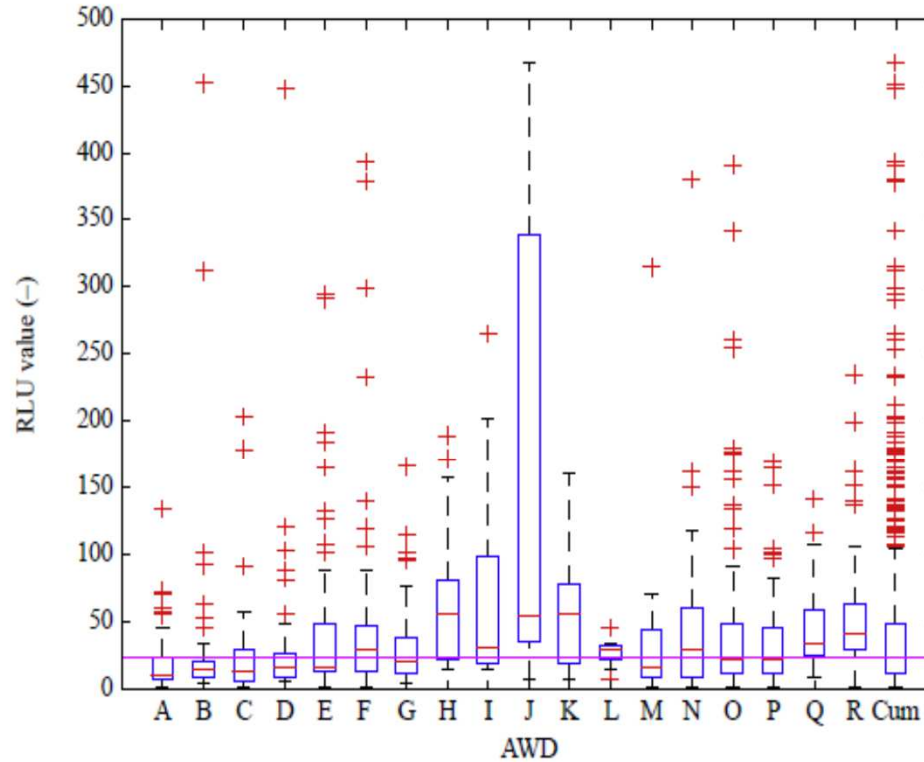**Figure 2.** Results of all 18 automated washer-disinfectors (AWDs) (A–R) and the cumulative (Cum) results of all 18 AWDs presented in a box plot. The y-axis represents relative light unit (RLU) values from 0 to 500. The mean of the cumulative results was 24 RLU and is denoted by the magenta horizontal line. Eleven of the 880 processes had an RLU value >500 and were distributed over eight AWDs (E, D, H, I, J, L, O, and Q).

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